














SIC Guided Surgery – Drill Sequence and Surgical Planning Dimensions

| Drills | Counter-sinks | 2.0 mm | 2.8 mm | 3.1 mm | 3.25 mm | 3.75 mm | 4.25 mm | optional Bone Tap |
|--|---------------|--------|--------|---|---|--|---|---|
| Implants Ø | | | |  |  |  |  |  |
|  3.0 mm | | ✓ | ✓ | | | | | 3.0 mm |
|  3.4 / 3.7 mm | 3.4 mm | ✓ | ✓ S | N | H | | | 3.4 mm |
|  4.0 / 4.2 mm | 4.0 mm | ✓ | ✓ | ✓ S | N | H | | 4.0 mm |
|  4.5 / 4.7 mm | 4.5 mm | ✓ | ✓ | | ✓ S | N | H | 4.5 mm |
|  5.0 / 5.2 mm | 5.0 mm | ✓ | ✓ | | ✓ | ✓ S | N | 5.0 mm |

 Soft  Normal  Hart

Drill template and instruments



Application through drill guides



Application through template